



Cleaning Instructions

Purpose:

This document was prepared to provide instructions for the care and cleaning for all DataCart mobile cart solutions. A stringent program of cleaning and disinfecting has been proven to reduce the incidence of hospital acquired infections.

Important Information / Recommendations for Use:

Detergents:

- Detergents Low foaming detergents with a pH range between 6.0 and 8.0 are recommended. Detergents with a pH outside this range can have an adverse effect or be damaging to some product.
- Enzymatic detergents aid in the removal of organic soil such as blood.
- Detergents should be used at the concentration and temperature recommended by the detergent manufacturer. There are some alkaline detergents that have been formulated to be safe for reprocessing medical devices. Refer to the manufacturer's limitations and warnings for information concerning specific materials that are adversely affected by the detergents.

Following is a list of detergents that DataCarts can be safely cleaned and disinfected with:

- Asepticare TB-II
- Bleach 5:1
- Bleach-Rite® disinfecting spray
- Cavicide® surface disinfectant
- Citrace® germicide
- Clorox® germicidal wipes
- Dispatch® spray hospital cleaner disinfectant with bleach
- Fade-A-Dyne® stain remover
- Lysol® spray disinfectant
- Precise™ hospital cleaner
- PDI® Super Sani-Cloth® germicidal wipes
- PDI Sani-Cloth HB germicidal wipes
- PDI Sani-Cloth Plus germicidal wipes
- SaniZide Plus® germicidal solution
- Virex® II 256 cleaner
- Virox® AHP 5 disinfectant cleaner surface wipes

Manual Cleaning Tools:

- Manual cleaning tools necessary for manual cleaning include: soft low linting cloths, cotton tip applicators, and non-abrasive brushes.
- Do not use abrasive cleaning tools (i.e. scouring pads or metal brushes).
 - Cleaning tools must be cleaned and inspected between use. Cloths should be clean and lint free and changed frequently. Brushes should be clean. Discard worn brushes and disposable cleaning tools.

Please contact our Customer Service Department at 847-459-1460 with any additional questions.



3175 MacARTHUR BLVD. • NORTHBROOK, IL • 60062

847-459-1460 • FAX 847-714-9443

www.lund-industries.com

www.data-carts.com