

*DATA*CART™

by LUND

Product Manual



Models: DCT-1K Series

Version 1.6
Jan, 2022

Statement of Use

Each *DataCart™* is designed to set a high standard in quality, ease of use, and customer satisfaction. Our goal is to provide a product that is built to exacting standards and is ready for years of durable service.

TESTED TO COMPLY WITH:

- UL Standard for Safety Medical Equipment, Part 1: General Requirements for Safety

RECOMMENDED SHIPPING AND OPERATING CONDITIONS:


- Temperature Range 25°F- 120°F / Humidity Range 10% - 80%

TERMS OF USE: Removing or damaging the serial number will void all cart warranties.

For maximum stability, lower the *DataCart™* completely when rolling over thresholds or other uneven surfaces.

Warranty Information: <https://data-carts.com/product-downloads/>

Cart Features

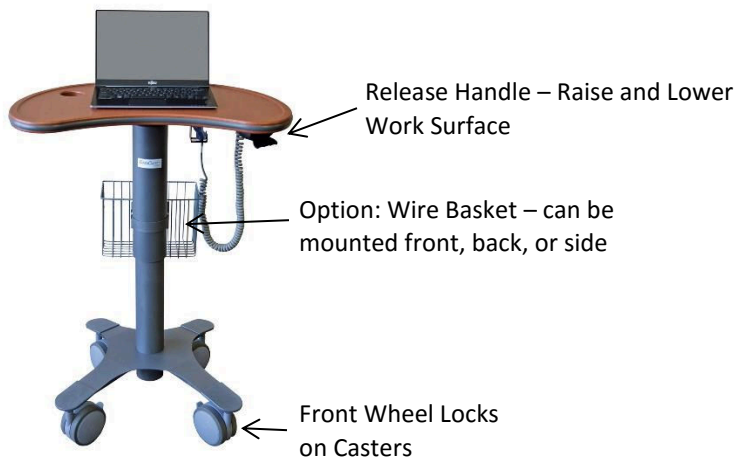

Additional Options:

- Vital Signs Monitor Mount
- Accessory Arm Interface Bracket
- Printer / Scanner Shelves
- Cup Holders
- Scanner Holders
- Sani Wipe Holder
- Wire Baskets



DataCart™ Optional Storage Drawers

Combi-Cam Lock Combination Instructions

- On the underside of the outer edge of the lock, there is a hole for the code reset button. Rotate the face of the lock until the code reset button comes into view.
- Push and hold this pin in with a paper clip.
- Set your personal combo by turning the dials to desired 3-digit combo.
- Release the reset button with the paperclip and your combo is set.
- Be sure to save your new combo; rotate the lock to the "Locked Position". Rotate one or more dials after lock is closed. To open the lock, turn the dials to your personal combination you just set.

DataCart™ Height Adjustment

The height adjustment handle is located on the right side of the work surface.

- Adjust the height up and down by pressing the handle or paddle upwards.

DataCart™ Locking Wheels

All wheels have locks to prevent the cart from moving.

- Push each lever down with your foot to engage the locks.
- Push each lever up with your foot to release the locks.

Technical Specifications

Work Surface: 28" W x 18" L; large – 32" W x 22" L

- Height Adjustment: 29.5" Low x 43" High

Wheelbase: 18" W x 17" L

Weight (Basic Cart Only): 58 lbs.

Warnings




Important Warnings



Electrical Shock Warning

The above symbols represent safety warnings that require significant attention when seen on the EMC cart or in the user manual. Failure to do so could result in minor injury, major injury, or even death.

 **SERVICE AND REPLACEMENT** Do not attempt to service or replace any part of the *DataCart™* unless directed to do so through *DataCart™* approved documentation (i.e., this User Manual or other instructions). If any component on the cart is missing or damaged, the cart must not be used. Contact your Reseller or *DataCart™* immediately to request service.



3175 MACARTHUR BLVD. • NORTHBROOK, IL • 60062
847-459-1460 • FAX 847-714-9443
www.lund-industries.com
www.data-carts.com



SAFE WORKING LOADS

- Safe working loads labels must be abided by for all parts



MAXIMUM WEIGHTS:

- Work Surface 20 lbs.



DO NOT USE THE UNIT IN OR NEAR WATER OR OTHER LIQUIDS

- If the unit becomes wet, unplug it immediately, wipe away any excess liquid and allow it to dry before use.



DO NOT TRANSPORT THE CART UP OR DOWN STAIRS



TRANSPORTING

- For maximum stability, lower the *DataCart*[™] completely when rolling over thresholds or other uneven surfaces.



COIL CORD

- Make sure the coil cord is secure prior to moving the cart
- When disconnecting, pull the plug rather than the cord itself
- Do not use if cord is damaged – contact Lund Industries for a replacement.



3175 MacARTHUR BLVD. • NORTHBROOK, IL • 60062

847-459-1460 • FAX 847-714-9443

www.lund-industries.com

www.data-carts.com